

# AHL Generic



## HIGH VISCOSITY TEMPORARY FILLING MATERIAL (New Packable Formula)

### DIRECTIONS FOR USE

#### FEATURES:

A ready to use temporary filling material with a synthetic resin base. Easily packable consistency with little or no tendency to stick to instruments. Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaptation. It is not a permanent filling material and recommended use is up to a maximum of 4 weeks only. Good sealing characteristic, easy to use consistency.

#### INDICATIONS:

- Temporary filling of cavities
- Temporary sealing of inlay preparations
- Temporary sealing of endodontic treatments
- Temporary fixing of post crowns

#### APPLICATION:

Using a suitable filling instrument, place direct into cavity and adapt as necessary. It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticatory pressure is not applied for approximately 1 hour. For provisional fixing of post crowns spread a layer on inside of crown and seat in normal way.

**NOTE:** For tube presentation, remove cap from tube, reverse and pierce membrane fully. Use key provided to assist extrusion from tube.

#### STORAGE:

Store in cool dry place (5 – 25°C). It is important that caps are tightly replaced after each use.

#### BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

#### SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 241X1 30g Jar



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

*Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.*

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2018-06

AP8469