

IMPRESSION PASTE

BS4284: 1981

DIRECTIONS FOR USE

A high definition material based on Eugenol (catalyst paste) and Zinc Oxide (base paste).

INDICATIONS:

The taking of impressions for the manufacture of dentures, particularly for edentulous patients.

The taking of impressions to enable mal-fitting dentures to be relined.

MIXING:

Squeeze equal lengths of material with the aid of a key from catalyst, and base paste tubes (approximately 3 inches each for a lower and 4 inches for an upper impression). Use a broad stiff spatula and mix thoroughly with sweeping strokes until a uniform colour indicates a completely homogeneous material (mixing time 30 seconds). Mix ratio: 2.75:1 by weight (base: catalyst). NB: Replace caps after extruding paste. Working time: 2 minutes at 23°C from end of mix.

IMPRESSION TAKING:

Apply mixed material to entire surface of a special tray (for denture manufacture) or to the thoroughly cleaned surface of existing denture (for refining). Setting time: 2 to 2 1/2 minutes at 37°C.

CLEANING:

Clean and disinfect the impression using Perform – ID (Schülke & Mayr) following the instructions on the packet. The material can be removed from spatulas using alcohol. (Caution: HIGHLY FLAMMABLE).

CAUTION:

This material contains Eugenol and like all such materials may occasionally cause the patient to experience short-lived mucosal irritation. Avoid contact with eyes - if this occurs, wash with copious quantities of water and seek medical attention.

STORAGE:

Store in a cool dry place.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 001X1 165g Base Paste/60g Catalyst Paste & Hand cream



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2012-10

L8078