

AHL Generic



LIGHT CURED GLASS IONOMER RESTORATIVE COATING

ISO 6874

DIRECTIONS FOR USE

FEATURES:

A light cure resin coating designed to protect and enhance glass ionomer restoratives. It is also beneficial to all other types of restorative materials.

INDICATIONS:

Transparent sealant for restorations to avoid immediate water and saliva contamination to exposed surfaces of glass ionomer cements.

CONTRA-INDICATION:

Do not use on patients with known allergies to resin based materials.

Do not use with Eugenol containing restorative materials.

If any sensitivity reactions are experienced, discontinue the use of the product and seek medical attention.

METHOD OF USE:

Place the glass ionomer according to manufacturer's instructions.

When restoration has set, dry surface to be coated do not desiccate

Dispense a few drops of the coating on to a suitable dish or mixing pad and replace the lid immediately.

Apply coating with a disposable applicator within 2 minutes of dispensing and cure immediately, using a dental surgery light at 470nm for 10 seconds.

Re-apply and cure following any subsequent adjustment or polishing.

Note:

1) Non-bur cut enamel should be treated with a 35-40% phosphoric acid according to the manufacturer's instructions. Rinse and dry with oil free air.

2) Place light source as close as possible to the coated surface.

3) If surface is tacky or yellowish, repeat light curing.

4) For use only by a dental professional in the recommended indications.

5) SDS available at www.ahl.uk.com.

SPECIAL NOTE:

Always replace cap immediately after use.

STORAGE:

Material is light sensitive. Store in a dark place at 5 - 25°C.

WARNING:

Irritating to eyes and skin. May cause sensitisation by skin contact.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

After contact with skin, wash immediately with plenty of soap and water.

Wear suitable gloves and eye/face protection.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY



420X1

5ml Bottle



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2018-03

AP9716