

LIGHT CURED GLASS IONOMER BASE LINER CEMENT (POWDER / LIQUID)

ISO 9917-2:2010 Class 3b (Light + Chemical Cure)

DIRECTIONS FOR USE

FEATURES:

- HEMA free – lower allergy risk.
- Self-adhesive to tooth structure – no primer required.
- High flexural strength.
- Improved bond strength to dentin and enamel.
- Mixes easily to smooth creamy texture.
- Contains a high level of fluoride.
- Radiopaque.

RECOMMENDED INDICATIONS:

- As a base or a liner.
- Glass ionomer/composite “sandwich” restorations.
- Deciduous cavity restoration.

CONTRA-INDICATIONS:

- Pulp capping – use calcium hydroxide to line cavities where remaining dentin may be thin. Do not use eugenol-containing materials.
- In rare instances, some people may exhibit sensitivity to resins. If any such reactions are experienced, discontinue use of product and seek medical assistance.

APPLICATION:

Powder : Liquid ratio	1 level scoop : 1 bubble-free drop 0.9g : 0.2g
Mixing time Light curing time (halogen, LED)	20 seconds 20 seconds
Working time in absence of light (at 23±1°C)	1 minute 30 seconds minimum
Setting time in absence of light (at 37±1°C)	6 minutes 0 seconds maximum

1. Prepare cavity and clean with polyacrylic acid toothcleanser in normal way. Line deep cavities with calcium hydroxide if appropriate.
2. Place powder and liquid on pad and mix thoroughly with flat bladed spatula to creamy consistency. Pack cement in to cavity and light cure the entire surface.
3. After light curing or at 7 minutes after placement, the material is hard enough to allow placement of the primary restorative material.


WARNING:


Avoid contact of powder or liquid with the eyes. In case of contact, wash thoroughly with water and obtain medical advice.

STORAGE:

Store in a cool, dry place away from light and heat (5-25°C). Do not use past expiry date.


BATCH CODE:

 The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

 The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 346XA2BL 15g Powder / 6ml Liquid

 Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.