

# AHL Generic

## PLAQUE DISCLOSING PELLETS

### DIRECTIONS FOR USE:

#### FEATURES:

- Pre-soaked pellets for convenient way to apply disclosing solution.
- Red shows plaque younger than 3 days; blue for older than 3 days.
- Mild taste of strawberry (red) or mint (blue).

#### RECOMMENDED INDICATIONS:

- To assist visual identification of plaque deposits on teeth.

#### APPLICATION:

1. Pick up a pellet using tweezers and rub over teeth surface. Avoid staining soft tissues as much as possible. Single use pellets - dispose of pellet after use.
2. Rinse mouth with water and spit out. Remaining plaque should now be stained.
3. Remove plaque using appropriate brush and paste.
4. Any residual staining (e.g. on gums) will disappear naturally overnight.

#### WARNING:

Do not allow pellets to contact skin or clothing - dye may stain if not rinsed off immediately.

#### STORAGE:

Store in a cool dry place (5-25°C). Close caps immediately after use.

#### BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

**REF:**        340X1                    Red Pellets x 100  
                  341X1                    Blue Pellets x 100



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK  
Tel: +44 1892 870500                    Fax: +44 1892 870482

*Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.*

2012-10

L6620