

PROPHYLAXIS PASTE

DIRECTIONS FOR USE

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INDICATIONS:

Use for the removal of plaque and tartar from patients teeth.

DIRECTIONS FOR USE:

The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use. Replace lid after use. For best results, use a prophy type handpiece and rubber cup / bristle brush combination. Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

PRECAUTIONS:

Replace lid after use.

STORAGE:

Store in a cool dry place (5-25 C)

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF:	014X1	Medium Grain Orange Oil Free	300g	
	018X1	Fine Grain Lemon Oil Free	300g	
	019X1	Coarse Grain Mint Oil Free	300g	
	033X1	Coarse Grain Orange Regular	250g	
	034X1	Medium Grain Orange Regular	250g	
	036X1	Medium Grain Mint Oil Free	300g	
	040X2	Medium Grain Mint Regular	250g	
	323X1	Coarse Grain Orange Regular	250g	<i>(For Hot Climate)</i>
	324X1	Medium Grain Mint Regular	250g	<i>(For Hot Climate)</i>
	325X1	Medium Grain Strawberry Regular	250g	<i>(For Hot Climate)</i>



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.