

## TEMPORARY DENTAL CEMENT

### DIRECTIONS FOR USE

#### FEATURES:

A ready to use temporary cement which re-fixes dental crowns, inlays and bridges. It is not a glue – it is a cement which sets only if it comes into contact with water or saliva and takes approximately 15-20 minutes to set. This cement has been developed to provide an ideal temporary solution and recommended use is up to a maximum of 4 weeks only. It does not irritate the gums, it is easy to work with and can be easily removed by a dentist without any damage to the bridge or crown. The composition has been tested by clinicians. To ensure best results, please read the instructions carefully.

#### INSTRUCTIONS FOR USE:

- Using a pin or similar object remove as much as possible of the old, loose cement from inside the bridge or crown. Do not use undue force that may damage the crown. Clean the inside of the crown with warm water and soap or detergent and rinse it under running water.
- Remove excess water with a cotton wool bud or tissue.
- Clean the area in the mouth (tooth stump) where the crown is to be replaced. Rinse the mouth with water (**do not dry the stump**).
- Apply a small amount of the cement around the inner edge of the crown or cap. Do not fill the crown. **For bridges**, apply a small amount of the cement around the **inner edge** of each crown of the bridge.
- Press the crown in place for 3 minutes. **For bridges** fit the bridge back in your mouth and firmly press down with your thumb and fingers. Then carefully bite into place. You will know when the crown or bridge is correctly re-seated as you can close your teeth back together again comfortably. (If you are not satisfied with the fit, you have up to 5 minutes to remove the bridge or crown and repeat the whole process).
- Remove any excess cement from around the base of the crown. **The setting of the cement takes approximately 15-20 minutes in contact with oral moisture. Do not chew for the next 3 hours**

#### IMPORTANT:

Please remember that the cement takes approximately 15-20 minutes to set in the presence of oral moisture – it is not a glue. Always use small quantities and apply to the inner edge of the crown(s). This product is a temporary solution – please consult your dentist as soon as possible.

#### STORAGE:

Store out of reach of children. Store in a cool, dry place (5-25°C / 41-77°F). It is important that the cap is replaced tightly after each use.

#### BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

#### SPECIALY FORMULATED FOR USE IN DENTISTRY

REF: 224X2 7g Tube



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

*Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.*

2018-06

AP8257