

TEMPORARY LUTING CEMENT

DIRECTIONS FOR USE:

FEATURES:

A ready to use temporary cement which fixes dental crowns and bridges. Cement sets in contact with water or saliva. It does not irritate the gums, it is easy to work with and can be easily removed without damage to the bridge or crown. It is not a permanent filling material and recommended use is up to a maximum of 4 weeks only.

INDICATIONS:

A temporary cement for fixing dental bridges, temporary crowns or bridges and all metal, porcelain and acrylic crowns.

INSTRUCTIONS FOR USE:

1. Clean out the crown or bridge (if necessary) and leave moist but not wet.
2. Clean the area in the mouth (tooth stump) where the crown is to be replaced. Rinse the mouth with water (**do not dry the stump**).
3. Apply a small amount of the cement around the inner edge of the crown or cap. Do not fill the crown. **For bridges**, apply a small amount of the cement around the **inner edge** of each crown of the bridge.
4. Press the crown or bridge in place for 3 minutes. (If you are not satisfied with the fit, you have up to 5 minutes to remove the bridge or crown and repeat the whole process).
5. Remove any excess cement from around the base of the crown. **The setting of the cement takes approximately 15 minutes in contact with oral moisture. Do not chew for the next 1 hour.**

STORAGE:

Store in a cool, dry place (5-25°C / 41-77°F). It is important that the cap is replaced tightly after each use.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALY FORMULATED FOR USE IN DENTISTRY

REF: 262X28G 4 x 7g Tube
262X8.4 3 x 2.8g Syringe Kit



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2018-06

AP8060