

TISSUE CONDITIONER MATERIAL

Non-Irritating, Soothing, Resilient and Self-Curing.

DIRECTIONS FOR USE

INDICATIONS:

As soft temporary denture reline material and tissue conditioner. As functional impression material.

1. Clean denture thoroughly by brushing with liquid detergent and water.
2. Inspect denture for proper occlusion.
3. Relieve all undercut and obvious pressure areas
4. Using syringe, place 4ml of liquid into the mixing cup provided. Add 1 level-full measuring vial of powder and mix gently with a stainless steel spatula for 30 seconds.
5. After allowing the mixture to form a viscous, yet flowable state, pour the mixture into the denture, using a spatula to flow the material to the peripheral border to be certain that there is ample material to form an adequate peripheral (muscle-trimming) roll.
6. Have the patient rinse their mouth thoroughly with plain cold water.
7. Place the denture in the mouth of the patient and have the patient close the jaw slowly to a normal biting position to bring the teeth into proper occlusion. Have the patient hold this position for 1-2 minutes before muscle trimming.
8. After a further 3 minutes, remove the denture from the mouth and check for full coverage and an adequate peripheral roll. Additional material can be added as required, as the material will blend homogeneously. If additional material is used, re-insert the denture for 2-3 minutes and re-inspect.
9. Trim excess with warm water and a sharp knife.
10. Have patient return periodically for examination of progress during the resorption period. Additional material can be added as required after grinding and relieving areas where the base material has shown through.
11. Tissue Conditioner can be readily removed from a denture by scraping with a blunt knife or spatula and then wiping the denture with alcohol before washing the denture with a mild detergent and warm water.
12. When Tissue Conditioner is used as a Functional Impression Material, allow the patient to wear and use the denture normally for 6-24 hours. Remove the impression from the mouth and pour the stone model immediately.

CLEANING:

Before sending dentures to a dental lab, always clean and disinfect them using Perform – ID (Schülke & Mayr) following the instructions on the packet.

WARNING:

Powder and liquid may cause slight irritation to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. The powder contains polymerised methacrylate and therefore may cause sensitisation by contact in susceptible persons. If sensitisation occurs, discontinue use.

STORAGE:

Store materials in a cool, dry place. Always replace all caps immediately after use.

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY.

REF: 131XPL 100g Powder/ 100ml Liquid



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.