

XEROCEM AUTOMIX EUGENOL-FREE TEMPORARY CROWN AND BRIDGE CEMENT

ISO 3107

DIRECTIONS FOR USE

FEATURES:

XeroCem is a totally eugenol-free cement based on zinc oxide and EBA and it dispenses easily from the automix syringe. It has a smooth creamy consistency with a long working time and a quick setting time. It is radiopaque with low film thickness and it does not inhibit resin polymerisation (light cure or self-cure).

RECOMMENDED INDICATIONS:

- Temporary cementing of provisional crowns and bridges.
- Patients with known allergy/sensitivity to eugenol.

CONTRA-INDICATIONS:

1. Pulp capping.
2. There are no known systemic side-effects to date. However some people may exhibit sensitivity to the material. If any such reactions are experienced, discontinue use of product and seek medical assistance.

APPLICATION:

Working Time (at 23 ± 1°C)	1 Minute 30 Seconds Minimum
Setting Time (at 37 ± 1°C)	3 Minutes 0 Seconds Maximum

1. Clean the preparation and restoration with water and dry. Use calcium hydroxide if pulp protection is required.
2. Remove cap from syringe. Bleed syringe on initial use. Place automix tip on to syringe and turn through 90° to lock in place. The material is now ready to be dispensed.
3. Spread a layer of cement over all inside surfaces of restoration and place on preparation using light pressure. Carefully trim excess cement when rubbery.

STORAGE:

Store in a cool dry place (5-25°C). Keep syringes capped when not in use.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 375X1 2 x 5ml Syringes with 6 Tips



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.