

ZOE IMPRESSION PASTE

BS 4284:1981

DIRECTIONS FOR USE

FEATURES:

A high definition material based on Eugenol (Catalyst Paste) and Zinc Oxide (Base Paste).

INDICATIONS:

The taking of impressions for the manufacture of dentures, particularly for edentulous patients. The taking of impressions to enable mal-fitting dentures to be relined.

MIXING:

Squeeze 8 to 10 cm (by weight 1 part red eugenol paste against 2.5 white zinc oxide paste) of paste from each tube onto mixing pad. Spatulate using wide spatula for 30 seconds or until smooth, creamy, uniformly coloured paste is obtained.

IMPRESSION TAKING:

Apply mix onto denture or a customized impression tray and place in mouth within 1 minute of end of mixing. Hold in mouth for at least 45 seconds.

Total working time: 2 min 15 sec.

Total mixing time: 3 min 15 sec.

The above quoted times apply when the ambient temperatures range is 20–25°C. Lower temperatures (e.g. when the paste is taken directly from cool place of storage) will give longer working and setting times.

CLEANING:

Clean and disinfect the impression using Perform-ID (Schülke & Mayr) following the instructions on the packet. The material can be removed from spatulas using alcohol. (Caution: HIGHLY FLAMMABLE).

WARNING:

This material contains Eugenol and like all such materials may occasionally cause the patient to experience short-lived mucosal irritation. Avoid contact with eyes. In case of contact, wash thoroughly with water and obtain medical advice.

STORAGE:

Store in a cool, dry place (5-25°C). Always replace cap after use, taking care not to interchange caps between the two paste tubes. Avoid storage of product above 25°C as this will cause oil separation from the paste and accelerated ageing of product.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: 473X1 120g Base Paste / 108g Catalyst Paste



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2017-02

AP9214