

AMALGOMER™

Technology

Dentine Conditioner

DIRECTIONS FOR USE

Amalgomer Dentine Conditioner is a solution of polyacrylic acid designed for removing the salivary pellicle on uncut or contaminated tooth surfaces thus enhancing the adhesion of Amalgomer or other glass ionomer restoratives.

METHOD OF USE:

Apply to surface using a pledget of cotton wool for a maximum of 30 seconds. If freshly cut dentine or enamel is contaminated with saliva, apply for 10 seconds only. Wash with water and dry with oil-free air. Do not desiccate the tooth. Fill immediately with Amalgomer or other glass ionomer restorative.

CONTRA-INDICATIONS:

Keep away from pulp cavity. Line deep cavities with a calcium hydroxide liner before using the Toothcleanser.

SPECIAL NOTICE:

Always replace cap tightly immediately after use.

WARNING:

Contains 25% polyacrylic acid - avoid contact with eyes. In case of contact, wash thoroughly with water and obtain medical advice.

STORAGE:

Store in a cool dry place.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

Specially formulated for use in dentistry.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

E-mail: sales@ahl.uk.com

Web: www.ahl.uk.com



Advanced Healthcare Ltd. operates a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

2017-04

AP8976

**THE STRENGTH
BEHIND YOUR SMILE**