

AMALGOMER™

Technology

Varnish (Light Cured)

DIRECTIONS FOR USE

Amalgomer Varnish is a light cure resin glaze designed to protect and enhance Amalgomer and other glass ionomer restorations. It is also beneficial to all other types of restorative materials.

METHOD OF USE:

After placement and initial setting of Amalgomer or other restorative apply a thin layer with a suitable brush and cure immediately, using a dental surgery light at 470nm for 10 seconds. Re-apply and cure glaze following any subsequent adjustment or polishing.

SPECIAL NOTE:

Always replace cap immediately after use.

WARNING:

Irritating to eyes and skin. May cause sensitisation by skin contact

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. After contact with skin, wash immediately with plenty of soap and water. Wear suitable gloves and eye/face protection.

STORAGE:

Store in a cool dry place.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

Specially formulated for use in dentistry.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500

Fax: +44 1892 870482

E-mail: sales@ahl.uk.com

Web: www.ahl.uk.com



Advanced Healthcare Ltd. operates a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

2017-04
AP8977

**THE STRENGTH
BEHIND YOUR SMILE**