

AHL

AHfil LCG

Light-Cure Glaze

DIRECTIONS FOR USE

FEATURES:

AHfil LCG is a light cure resin glaze designed to protect and enhance glass ionomer restoratives. It is also beneficial to all other types of restorative materials.

INDICATIONS:

- Transparent sealant for restorations.

CONTRA-INDICATIONS:

- Do not use on patients with known allergies to resin based materials.
- Do not use with Eugenol containing restorative materials.

METHOD OF USE:

After placement and initial setting of glass ionomer or other restorative apply a thin layer with a suitable brush and cure immediately, using a dental surgery light at 470nm for 10 seconds. Re-apply and cure glaze following any subsequent adjustment or polishing.

SPECIAL NOTE:

Always replace cap immediately after use.

WARNINGS:

Irritating to eyes and skin. May cause sensitisation by skin contact
In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. After contact with skin, wash immediately with plenty of soap and water. Wear suitable gloves and eye/face protection.

STORAGE:

Store at 5 - 25°C.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: AH0251 5ml Liquid



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2017-02

AP9262