

AHL

AHfil TC

Tooth Conditioner

DIRECTIONS FOR USE

FEATURES:

AHfil TC is a solution of polyacrylic acid designed for removing the salivary pellicle on uncut or contaminated tooth surfaces thus enhancing the adhesion of glass ionomer restoratives.

METHOD OF USE:

Apply to surface using a pledget of cotton wool for a maximum of 30 seconds. If freshly cut dentine or enamel is contaminated with saliva, apply for 10 seconds only. Wash with water and dry with oil-free air. Do not desiccate the tooth. Fill immediately with glass ionomer restorative.

CONTRA-INDICATIONS:

Keep away from pulp cavity. Line deep cavities with a calcium hydroxide liner before using the Toothcleanser.

SPECIAL NOTICE:

Always replace cap tightly immediately after use.

WARNING:

Contains 25% polyacrylic acid - avoid contact with eyes. In case of contact, wash thoroughly with water and obtain medical advice.

STORAGE:

Store in a cool dry place.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: AH0250 10ml Liquid



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2015-09

AP9157