

AHlute+

Resin Modified Glass Ionomer Luting Cement Capsules

ISO 9917-2 Class 1a.

DIRECTIONS FOR USE

FEATURES:

Radiopaque resin modified glass ionomer luting cement with high strength, excellent aesthetics and contains fluoride. Excellent adhesion to metal and tooth structure.

RECOMMENDED INDICATIONS:

- Cementing of all metal, ceramic, porcelain and resin crowns, bridges, inlays and onlays.
- Cementing of metal and ceramic orthodontic brackets to tooth structure.

CONTRA-INDICATIONS:

1. Pulp capping.
2. Some people exhibit sensitivity to resins. If any such reactions are experienced, discontinue use of product and seek medical assistance.

PREPARATION:

Clean the preparation and restoration with water and dry. Use calcium hydroxide if pulp protection is required. Do not use eugenol-containing materials.

MIXING:





1. Remove a capsule from an aluminium bag immediately before mixing. Tap gently on a hard surface to loosen any compacted powder.
2. Rotate the plunger clockwise until it stops securely (30°)(Fig.A).

• Ensure you do not lift the nozzle before mixing.

3. Press the plunger completely to ensure that all of the liquid is pressed into the powder (Fig.B)
4. Insert the capsule into the mixer and mix for 10 sec. (frequency: 4000 rpm or more). (Fig.C).

APPLICATION

1. After mixing, remove the capsule from the mixer immediately. Then, place it into the applicator.
 2. Lift the nozzle completely to dispense the paste (Fig.D).
- If the nozzle is not lifted completely, the paste cannot be dispensed correctly.
3. Dispense a layer of cement over all inside surfaces of restoration and place on preparation. Trim excess when cement is rubbery.

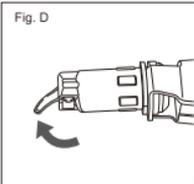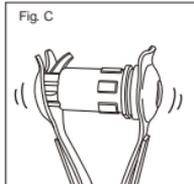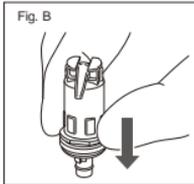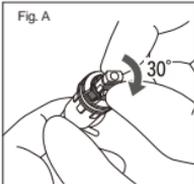

DO NOT LIFT THE NOZZLE BEFORE MIXING



The capsule may only be used once due to hygienic reasons (prevention of cross-contamination between patients).

STORAGE:

Store in a cool, dry place (5-25°C).

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

Average contents per capsule: 0.32g Powder and 0.20g Liquid.

Minimum net volume of mixed cement per capsule: 0.46g

SPECIALLY FORMULATED FOR USE IN DENTISTRY



AH0510

50 Capsule



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.