

Tfil eco

Temporary Filling Material

DIRECTIONS FOR USE

FEATURES:

A ready to use temporary filling material with a zinc oxide base. Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaptation. Good sealing characteristic, easy to use consistency. It is not a permanent filling material and recommended use is up to a maximum of 4 weeks only.

INDICATIONS:

Temporary filling of cavities
Temporary sealing of endodontic treatments

APPLICATION:

Using a suitable filling instrument, place direct into cavity and adapt as necessary. It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticatory pressure is not applied for approximately 1 hour.

STORAGE:

Store in cool dry place (5 – 25°C). It is important that caps are tightly replaced after each use.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.




The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: AH1020 38g Jar (White)



 Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

2018-06

AP9160