

AHL

caviLINE

Calcium Hydroxide Base Liner

DIRECTIONS FOR USE

FEATURES:

caviLINE is a rigid, self-setting, radiopaque composition that will not inhibit the polymerisation of acrylic and composite restorations.

INDICATIONS:

Direct and indirect pulp capping.

Protective liner under dental filling materials, cements and other base materials.

CONTRAINDICATIONS:

Hypersensitivity against sulphonamides or other components, respectively of the product.

Acetone-containing products are detrimental to calcium hydroxide materials. Minimise contact of these products with calcium hydroxide liners.

MIXING:

Extrude equal volumes of the base and catalyst paste onto a mixing pad. Mix immediately using a spatula until a uniform colour is achieved. **DO NOT OVER SPATULATE.** To provide maximum working time, complete mixing of the two pastes within 10 seconds.

APPLICATION:

Quickly apply the mixed material into the cavity. For efficient placement before setting starts. The cavity preparation should be dry. Avoid touching the margins of the cavity. Avoid placing a large bulk of material.

The mixed material will set in 2 to 4 minutes in the mouth. Increased humidity and elevated temperature will reduce available work time and set time. Conversely, decreased humidity and lower temperature will increase available work time and set time. Remove any excess set material from retention areas or margins with a sharp spoon excavator or bur.

STORAGE:

Store in a cool, dry place (5-25°C / 41-77°F). It is important to keep tubes tightly sealed when not in use. Separation of solid and liquid phases of the base paste is accelerated at elevated temperatures. When stored at reduced temperatures, allow the material to return to normal room temperature before using.

WARNING:

May cause allergic reactions in persons susceptible to sulphonamides.

Contains calcium hydroxide. Avoid contact with skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of accident or if you feel unwell, seek medical advice immediately (show label where possible).

Base paste contains 1,3-Butylene Glycol Disalicylate. Irritating to skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. In case of contact with skin, wash immediately with plenty of soap and water.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.




The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: AH0760 13g Base Paste / 11g Catalyst Paste



 Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2017-07

AP9172