

AHL

proKlene & proKlene+

Prophylaxis Paste

DIRECTIONS FOR USE

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INDICATIONS:

Use for the removal of plaque and tartar from patients teeth.

DIRECTIONS FOR USE:

The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use. Replace lid after use. For best results, use a prophy type handpiece and rubber cup / bristle brush combination. Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

PRECAUTIONS:

Replace lid after use.

STORAGE:

Store in a cool dry place (5-25°C)

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: Regular: proKlene

AH1400	Prophylaxis Paste Regular Coarse Orange 250g
AH1401	Prophylaxis Paste Regular Medium Mint 250g
AH1425	Prophylaxis Paste Regular Coarse Orange 50g
AH1426	Prophylaxis Paste Regular Medium Mint 50g

Oil Free: proKlene+

AH1450	Prophylaxis Paste Oil Free Coarse Mint 250g
AH1451	Prophylaxis Paste Oil Free Medium Orange 250g
AH1452	Prophylaxis Paste Oil Free Fine Lemon 250g
AH1475	Prophylaxis Paste Oil Free Coarse Mint 50g
AH1476	Prophylaxis Paste Oil Free Medium Orange 50g
AH1477	Prophylaxis Paste Oil Free Fine Lemon 50g



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2015-01

AP9291