

AHL

reliaBond Xtra

Light Cure Self-Etching One Step Dental Adhesive

DIRECTIONS FOR USE

FEATURES:

A HEMA-free "all-in-one adhesive" to etch, prime and bond in one step. It features ease of use and excellent bond strength to enamel and dentine. Bond durability and biocompatibility are outstanding. Suitable for use with composites and compomers.

INDICATIONS:

- Suitable for bonding composites and compomers to enamel, dentine and metal.

CONTRA-INDICATION:

Exposed pulp, pulpitis, known allergy to methacrylate resin.

SIDE EFFECTS:

Isolated cases of contact-allergies with products of similar composition have been reported. To avoid possible pulpal reactions, protect the exposed dentine in the pulpal area with an appropriate lining material (preferably with a calcium hydroxide cavity liner).

INTERACTIONS:

Avoid cavity liners which might inhibit polymerisation due to their components, e.g. phenolic substances such as zinc oxide eugenol.

APPLICATION:

1. Clean the tooth surface and prepare the cavity following conventional methods. Isolate with rubber dam if appropriate.
2. Apply calcium hydroxide cavity liner in deep cavities where remaining dentine is thin.
3. Apply a thin layer of adhesive and leave undisturbed for 10 seconds. Dry gently with oil-free air for 5 seconds.
4. Light cure with dental curing light; halogen 10 seconds, LED 5 seconds. Cavity is now ready for filling with composite or compomer restorative.

STORAGE:

Store in a refrigerator (1-10°C).

Caution: Do not expose to direct sun light or other heat sources. Do not freeze.

WARNING:

Close bottle after each use. Keep material out of the reach of children. In case of direct contact with the oral mucosa, rinse with plenty of water. Harmful if swallowed. Irritating to eyes and skin. May cause sensitisation by skin contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. After contact with skin, wash immediately with plenty of soap and water. Wear suitable protective clothing, gloves and eye/face protection. Highly flammable liquid.

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

SPECIALLY FORMULATED FOR USE IN DENTISTRY

REF: AH1100 6ml Light-Cure Self-Etching Adhesive & Accessories
AH1160 6ml Light-Cure Self-Etching Adhesive Bottle



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482

Advanced Healthcare Ltd operate a policy of continuous monitoring and improvement of our products. If you have any comments about this product, please contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



2017-11

AP8841