



**FEEDBACK**  
www.ahl.uk.com/feedback

# AHL

## proRinse

Concentrated Mouthrinse

### DIRECTIONS FOR USE

#### FEATURES:

A super concentrated mouth rinse to be diluted in water.

#### INTENDED PURPOSE:

Removal of post treatment debris

#### INTENDED PATIENT POPULATION:

From child to geriatrics

#### INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

#### CLINICAL BENEFIT:

Removal of post treatment debris.

#### INDICATIONS FOR USE:

- For the freshening of the patient's mouth.
- For the mechanical removal of post treatment debris.

#### CONTRA-INDICATIONS:

None

#### CONTENTS OF PACK:

100ml Liquid

#### PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.
- Do not use undiluted.

#### PROCEDURE

##### (1) APPLICATION:

- Dispense one dose of concentrate into an empty beaker
- Fill beaker with water
- Patient should rinse mouth with diluted product and spit it out

#### STORAGE:



Store in a cool, dry place (5-25°C).  
Always replace cap immediately after use.

#### EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

#### DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

#### POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

#### BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

#### DEVICE CODES:

**REF** AH1500 100ml Liquid

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

 Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK  
Tel: +44 1892 870500 Email: sales@ahl.uk.com

**EC REP** Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

2025-03

AP10216/2



**FEEDBACK**  
www.ahl.uk.com/feedback

# AHL

## proRinse

Concentrated Mouthrinse

### DIRECTIONS FOR USE

#### FEATURES:

A super concentrated mouth rinse to be diluted in water.

#### INTENDED PURPOSE:

Removal of post treatment debris

#### INTENDED PATIENT POPULATION:

From child to geriatrics

#### INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

#### CLINICAL BENEFIT:

Removal of post treatment debris.

#### INDICATIONS FOR USE:

- For the freshening of the patient's mouth.
- For the mechanical removal of post treatment debris.

#### CONTRA-INDICATIONS:

None

#### CONTENTS OF PACK:

100ml Liquid

#### PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.
- Do not use undiluted.

#### PROCEDURE

##### (1) APPLICATION:

- Dispense one dose of concentrate into an empty beaker
- Fill beaker with water
- Patient should rinse mouth with diluted product and spit it out

#### STORAGE:



Store in a cool, dry place (5-25°C).  
Always replace cap immediately after use.

#### EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

#### DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

#### POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

#### BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

#### DEVICE CODES:

**REF** AH1500 100ml Liquid

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

 Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK  
Tel: +44 1892 870500 Email: sales@ahl.uk.com

**EC REP** Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

2025-03

AP10216/2