



AHL Generic

PROPHYLAXIS PASTE

DIRECTIONS FOR USE

FEATURES:

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INTENDED PURPOSE:

Removal of plaque, tartar and scale from the tooth surface

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

Aid in the prevention of dental caries and help maintain the integrity of remaining teeth.

INDICATIONS FOR USE:

Use for the removal of plaque, tartar and scale from the tooth surface.

CONTRA-INDICATIONS:

None

CONTENTS OF PACK:

Prophy Pot

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes, and skin.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

SPECIAL NOTICE:

- The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use.
- Replace lid after use.
- For best results, use a prophy type handpiece and rubber cup / bristle brush combination.

PROCEDURE

(1) APPLICATION:

- Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus.
- Repeat this procedure for each tooth until all exposed surfaces have been treated.
- Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

DEVICE CODES:



014X1	Medium Grain Orange Oil Free	300g
019X1	Coarse Grain Mint Oil Free	300g
032X1	Coarse Grain Mint Regular	250g
033X1	Coarse Grain Orange Regular	250g
034X1	Medium Grain Orange Regular	250g
036X1	Medium Grain Mint Oil Free	300g
039X1	Coarse Grain Orange Oil Free	300g
040X2	Medium Grain Mint Regular	250g
041X2	Medium Grain Thymol Regular	300g
114X2	Medium Grain Citrus Regular	300g
303X3	Medium Grain Bubblegum Regular	250g
304X3	Medium Grain Strawberry Regular	250g
408X2	Mint Regular	200g

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database. <https://ec.europa.eu/tools/eudamed>

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email: sales@ahl.uk.com



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

2025-02

AP8336/3



FEEDBACK
www.ahl.uk.com/feedback