



## TOOTHCLEANSER

### DIRECTIONS FOR USE

#### FEATURES:

Toothcleanser is a solution of polyacrylic acid designed for removing the salivary pellicle on uncut or contaminated tooth surfaces thus enhancing the adhesion of glass ionomer restoratives.

#### INTENDED PURPOSE:

Removal of salivary pellicle

#### INTENDED PATIENT POPULATION:

From child to geriatrics

#### INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

#### CLINICAL BENEFIT:

Aid in the restoration of the function of teeth and help maintain the integrity of remaining tooth structure.

#### INDICATIONS FOR USE:

- Removal of the salivary pellicle on uncut or contaminated tooth surfaces

#### CONTRA-INDICATIONS:

- Keep away from pulp cavity.
- Line deep cavities with a calcium hydroxide liner before using the Toothcleanser.

#### CONTENTS OF PACK:

Liquid 10ml, instructions for use

#### PRECAUTIONS AND WARNINGS:

- Contains 25% polyacrylic acid - avoid contact with eyes.
- In case of contact, wash thoroughly with water and obtain medical advice.
- Do not expose patients or users known to be allergic to this type of material.

#### PROCEDURE

##### (1) APPLICATION:

- Apply to surface using a pledget of cotton wool for a maximum of 30 seconds.
- If freshly cut dentine or enamel is contaminated with saliva, apply for 10 seconds only.
- Wash with water and dry with oil-free air.
- Do not desiccate the tooth.
- Fill immediately with glass ionomer restorative.

#### STORAGE:



Store in a cool, dry place (5-25°C).  
Always replace cap immediately after use.

#### EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

#### DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

#### POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

#### BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

#### DEVICE CODES:



021X1 10ml Liquid

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK  
Tel: +44 1892 870500 Email: sales@ahl.uk.com



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.