



COCOA BUTTER

DIRECTIONS FOR USE

FEATURES:

To enhance the surface properties of glass ionomer cements. The thixotropic properties are carefully controlled to ensure easy placement.

INTENDED PURPOSE:

Restoration surface protection

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

To enhance the surface properties of glass ionomer cements.

INDICATIONS FOR USE:

- For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 1 hours after placement.
- To prevent cement from sticking to adjacent teeth and instruments.

CONTRA-INDICATIONS:

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and seek medical advice.

CONTENTS OF PACK:

Tube of cocoa butter.

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with the eyes.
- In case of contact, rinse thoroughly with large quantities of water and obtain medical advice as soon as possible.

PROCEDURE

(1) APPLICATION:

- Place glass ionomer cement according to the manufacturer's instructions.
- Extrude the required amount of material from the tube.
- Using a spatula or cotton pellet, apply uniformly to the area to be moisture proofed / isolated.
- After use, immediately replace cap.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

DEVICE CODES:

REF	330X1	1g	tube
	330X2	10g	tube

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email:sales@ahl.uk.com

EC REP Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.