



FEEDBACK
www.ahl.uk.com/feedback

AHL

proKlense

Mouthwash Tablets

DIRECTIONS FOR USE

FEATURES:

Mouth rinse tablets that rapidly dissolve in water.

INTENDED PURPOSE:

Removal of post treatment debris

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

Removal of post treatment debris.

INDICATIONS FOR USE:

- For the freshening of the patient's mouth.
- For the mechanical removal of post treatment debris.

CONTRA-INDICATIONS:

None

CONTENTS OF PACK:

1000 tablets

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.
- Do not ingest undiluted tablets

PROCEDURE

(1) APPLICATION:

- Dissolve one tablet in half a tumbler of warm water.

ACTIVE INGREDIENTS:

Sodium benzoate, Potassium benzoate, Spearmint Oil, Saccharin, Effervescent Base.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



DEVICE CODES:

REF AH1510 100ml Liquid

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email:sales@ahl.uk.com

EC REP

Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

2025-02

AP10215/2



FEEDBACK
www.ahl.uk.com/feedback

AHL

proKlense

Mouthwash Tablets

DIRECTIONS FOR USE

FEATURES:

Mouth rinse tablets that rapidly dissolve in water.

INTENDED PURPOSE:

Removal of post treatment debris

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

Removal of post treatment debris.

INDICATIONS FOR USE:

- For the freshening of the patient's mouth.
- For the mechanical removal of post treatment debris.

CONTRA-INDICATIONS:

None

CONTENTS OF PACK:

1000 tablets

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.
- Do not ingest undiluted tablets

PROCEDURE

(1) APPLICATION:

- Dissolve one tablet in half a tumbler of warm water.

ACTIVE INGREDIENTS:

Sodium benzoate, Potassium benzoate, Spearmint Oil, Saccharin, Effervescent Base.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



DEVICE CODES:

REF AH1510 100ml Liquid

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email:sales@ahl.uk.com

EC REP

Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

2025-02

AP10215/2