



FEEDBACK
www.ahl.uk.com/feedback

AHL

proKlene & proKlene+

Prophylaxis Paste

DIRECTIONS FOR USE

FEATURES:

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INTENDED PURPOSE:

Removal of plaque, tartar and scale from the tooth surface

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

Aid in the prevention of dental caries and help maintain the integrity of remaining teeth.

INDICATIONS FOR USE:

Use for the removal of plaque, tartar and scale from the tooth surface.

CONTRA-INDICATIONS:

None

CONTENTS OF PACK:

Prophy Pot

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes, and skin.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

SPECIAL NOTICE:

- The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use.
- Replace lid after use.
- For best results, use a prophy type handpiece and rubber cup / bristle brush combination.

PROCEDURE

(1) APPLICATION:

- Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus.
- Repeat this procedure for each tooth until all exposed surfaces have been treated.
- Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

DEVICE CODES:



Regular: proKlene

AH1400 Prophylaxis Paste Regular Coarse Orange 250g
AH1401 Prophylaxis Paste Regular Medium Mint 250g

AH1425 Prophylaxis Paste Regular Coarse Orange 50g
AH1426 Prophylaxis Paste Regular Medium Mint 50g

Oil Free: proKlene+

AH1450 Prophylaxis Paste Oil Free Coarse Mint 250g
AH1451 Prophylaxis Paste Oil Free Medium Orange 250g

AH1475 Prophylaxis Paste Oil Free Coarse Mint 50g
AH1476 Prophylaxis Paste Oil Free Medium Orange 50g



AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in..

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email:sales@ahl.uk.com



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta

2025-03

AP9291/2



FEEDBACK
www.ahl.uk.com/feedback

AHL

proKlene & proKlene+

Prophylaxis Paste

DIRECTIONS FOR USE

FEATURES:

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INTENDED PURPOSE:

Removal of plaque, tartar and scale from the tooth surface

INTENDED PATIENT POPULATION:

From child to geriatrics

INTENDED USER:

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

CLINICAL BENEFIT:

Aid in the prevention of dental caries and help maintain the integrity of remaining teeth.

INDICATIONS FOR USE:

Use for the removal of plaque, tartar and scale from the tooth surface.

CONTRA-INDICATIONS:

None

CONTENTS OF PACK:

Prophy Pot

PRECAUTIONS AND WARNINGS:

- Do not expose patients or users known to be allergic to any of the ingredients of this material.
- Avoid contact with eyes, and skin.
- In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

SPECIAL NOTICE:

- The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use.
- Replace lid after use.
- For best results, use a prophy type handpiece and rubber cup / bristle brush combination.

PROCEDURE

(1) APPLICATION:

- Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus.
- Repeat this procedure for each tooth until all exposed surfaces have been treated.
- Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

STORAGE:



Store in a cool, dry place (5-25°C).
Always replace cap immediately after use.

EXPIRY:



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL:

Dispose of the contents and containers in accordance with relevant local and national requirements.

POSSIBLE SIDE EFFECTS / RESIDUAL RISKS:

- This product contains substances that may cause an allergic reaction.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

DEVICE CODES:



Regular: proKlene

AH1400 Prophylaxis Paste Regular Coarse Orange 250g
AH1401 Prophylaxis Paste Regular Medium Mint 250g

AH1425 Prophylaxis Paste Regular Coarse Orange 50g
AH1426 Prophylaxis Paste Regular Medium Mint 50g

Oil Free: proKlene+

AH1450 Prophylaxis Paste Oil Free Coarse Mint 250g
AH1451 Prophylaxis Paste Oil Free Medium Orange 250g

AH1475 Prophylaxis Paste Oil Free Coarse Mint 50g
AH1476 Prophylaxis Paste Oil Free Medium Orange 50g



AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging. If you experience any serious incident relating to the use of this product, please immediately contact AHL at the below address and the competent authority of the territory you are in.

A summary of safety & clinical performance (SSCP) is available via the EUDAMED database.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Email:sales@ahl.uk.com



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta

2025-03

AP9291/2